

skin condition

Balanced to mature or prematurely-aging skin.

description

This unique treatment uses plasma technology to tighten and firm specific areas of the face or body.

benefits

- Firms and tightens skin
- Reduces the apparent depth of dynamic and set wrinkles
- Smooths skin

featured professional products

- PreCleanse
- Special Cleansing Gel
- Pro Restore



pro fibroblast

how it works**1. Preparation**

A double cleanse, followed by the application of Pro Restore helps support skin's barrier to create an optimal environment for fibroblast skin tightening.

2. Fibroblast

Specific areas are targeted and treated with fibroblast skin tightening.

3. Post-treatment care

After 24 hours, Pro Restore is applied 2 to 3 times daily to support's skin recovery and amplify treatment results.

treatment protocol**double cleanse**

1. PreCleanse
2. Perform Face Mapping.
3. Apply Special Cleansing Gel for the second cleanse.

pre-application

Apply a thin layer of Pro Restore and allow to absorb completely. Wipe the skin with damp esthetic wipe.

numbing

1. Apply numbing cream per manufacturer's directions and recommended time.
2. Remove numbing cream as directed by manufacturer. Use Special Cleansing Gel to assist with removal if a full cleanse is necessary.

skin tightening

Proceed to treat the selected area with fibroblast skin tightening.

dermal layering

1. Send client home with 2 tubes of Pro Restore for use.
 - a. Start 24 hours after the treatment.
 - b. Use morning and evening for 7 days.
2. Share additional at-home care instructions and recommended treatment program.